

Questions / Answers

What is happening during a CRISTAL® session?

First, your doctor will apply a CRISTAL® cryoprotective gel pad to the treatment area. Next, the applicator will be placed over top of the gel pad ; suction and cooling will begin. After the treatment, your doctor will massage the treated area for about ten minutes, in order to accelerate the lymphatic drainage process and the elimination of fat cells. Treatment may also be supplemented with an LED session, to help regenerate the skin.

How long does a CRISTAL® session last?

A session with the Cristal® takes 60 minutes per zone. You can choose to enjoy your session by reading or listening to music.

Can I carry on with my normal activities after a session?

There is no downtime required after treatment with the Cristal®, as it is non-invasive. You can continue with your normal activities immediately after the session.

When will I see results?

There are no immediate results, as the fat cells are gradually eliminated by the immune system. You will begin to see results at about 6-10 weeks after the treatment and you will continue to lose fat in the treated area during the following months.

Is the CRISTAL® medical Cryolipolysis safe? Is it painful?

Are there side effects?

CRISTAL® therapy is non-invasive, painless, and perfectly safe. This technique selects fat cells and does not affect any other surrounding tissue. It is possible to experience light side effects such as redness, bruising, swelling and a feeling of numbness in the treated area. These effects naturally disappear within a few hours or in a few days. There is no downtime required and you can continue with your normal activities after the treatment.

How many sessions are needed?

The results vary from patient to patient, but it is generally recommended to conduct one to two sessions to get the results you desire.

Are the results permanent?

Yes, the results are permanent, as the fat is permanently removed from your body. However it is important that you take good care of your new body with a balanced diet and a regular physical activity. Other complementary methods may also be used to further reduce adipose tissue.

Book an appointment with
your doctor today for
a CRISTAL® treatment:


For more information :
www.cristalcryolipolysis.com



CRISTAL®

Medical Cryolipolysis

Crystallise fat to redefine
your curves

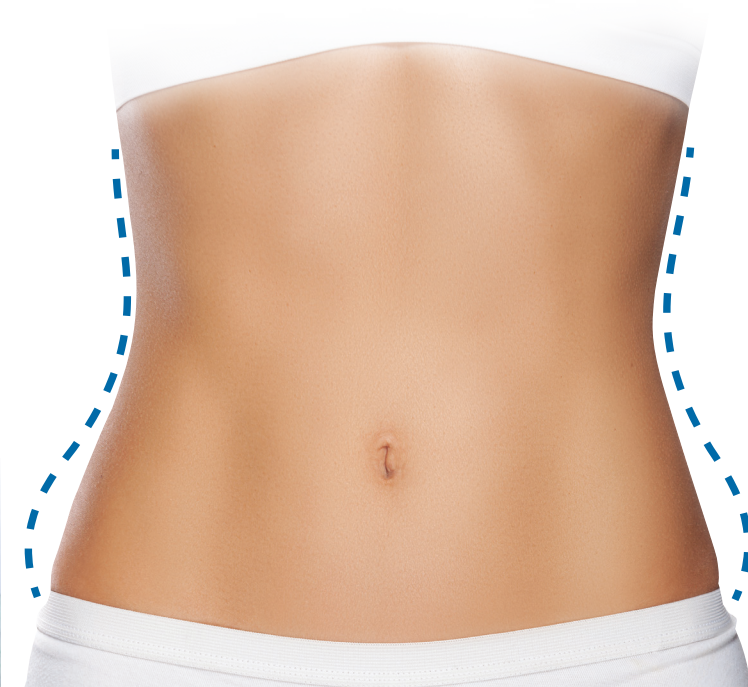

Leaflet-Patient-CRISTAL-24112017

This device was developed and manufactured in France by Deleo.

It is a Class IIa Medical Device. This machine is legalized with the CE marking that indicates that this product meets the essential requirements of all relevant European medical device directives. Delivered to Deleo by the notified body BSI. Please read instructions before use. Do not use the device other than specified in the instructions. If in doubt about the use / against-indications, you should first consult your doctor.

- Permanent results
- Proven effective
- Non invasive & no anesthesia
- Safe and secure system

BY Deleo
CREATOR IN
AESTHETIC MEDICINE



The ideal solution for a harmonious silhouette in all simplicity !

What is Medical Cryolipolysis?

This procedure crystallises fat cells, which then self-destruct due to the cold temperature applied.

The crystallised fat is removed naturally and permanently by your body in the weeks following the session, giving way to enhanced curves.

This innovative technique is safe and effective; it is non-invasive and requires no anesthesia.

You can continue with your normal activities immediately after the session.

CRISTAL® offers a range of several different applicators to handle all the curves of the body, for both men and women:

- Chin
- Arms
- Bra fold
- Abdomen
- Love handles
- Saddle bags
- Thighs
- Knees
- Calves

After 8 weeks, the result will be optimal and definitive; you will be able to enjoy your new curves!

CRISTAL® is the reference in medical cryolipolysis. Don't wait to find out!

Is the CRISTAL® treatment for you?


Yes, if you are looking for a gentle and effective method to safely harmonise your natural shape.

Yes, if you are looking for a comfortable method for losing your unsightly fat rolls.

Visible and permanent results




Copyright, Dr MELLUL

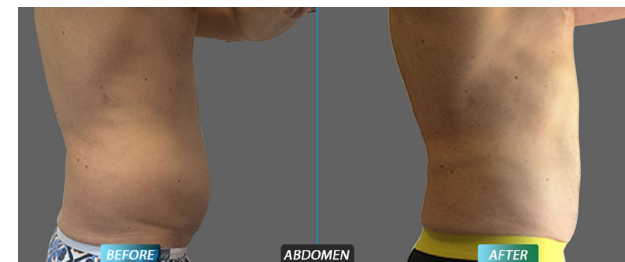

Copyright, Dr THANACODY



Copyright, Dr SCEMAMA

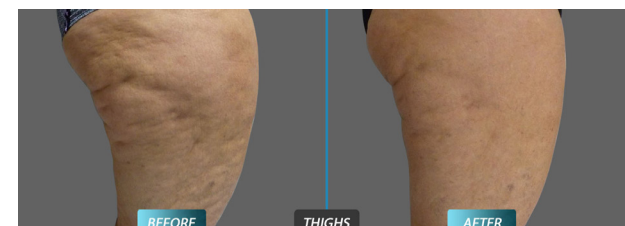

Copyright, Dr MALET